

MASIMO SAFETYNET ALERT™

Quick Start Guide

SYSTEM COMPONENTS



Masimo Chip™


- Reusable chip snaps into the sensor and communicates with the Bedside Station.



Masimo SafetyNet Alert App

- Displays Oxygen, Pulse Rate and other data.
- Cloud sends alerts to Bedside Station and Mobile device.
- Compatible with most mobile devices.

(mobile device not included)

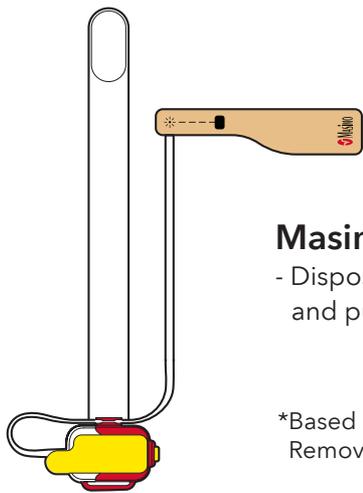

Masimo Sensor™


- Disposable sensor measures oxygen level and pulse rate and lasts 7 consecutive nights.*

*Based on 96 hours of sensor battery life. Remove chip between uses.



Bedside Station

- Collects data from Masimo Sensor.
- Sends data to smartphone app.

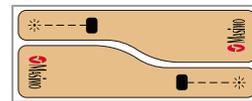

Replacement Tapes

- Additional tapes for replacement as needed.


Set Up App

1

1. Download app



2. Connect bedside station to power

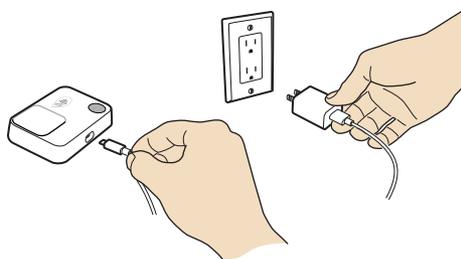

3. Look for power indicator light




4. Follow step-by-step instructions on app to set up system




Set Up Sensor

2


1. Remove sensor from pouch

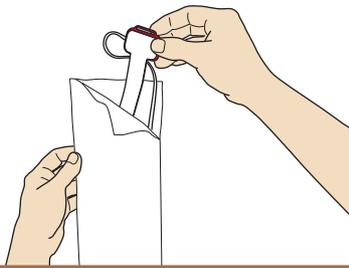


2. Peel off yellow label to activate battery

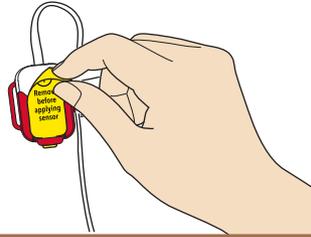

3. Place sensor on wrist as shown and carefully thread the tip of the strap through the open red loop

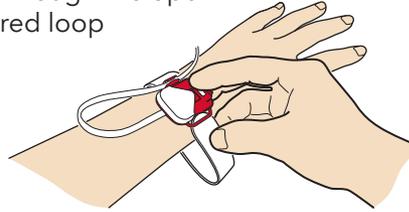


4. Wrap the loose end of the strap around your wrist and press

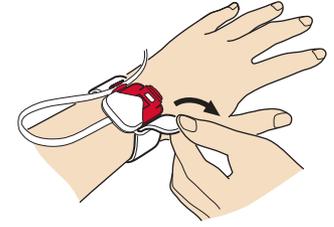


Apply Sensor

3


1. Remove part of the plastic film and place star (✱) on top of fingernail of ring finger

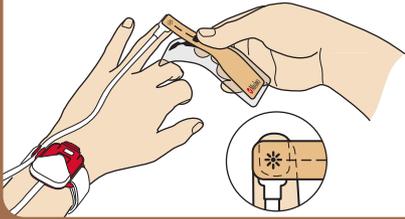


2. Wrap tape around finger so black square (■) is on finger pad directly across from star (✱)

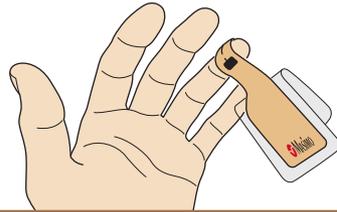

3. Squeeze top and bottom of finger to secure tape

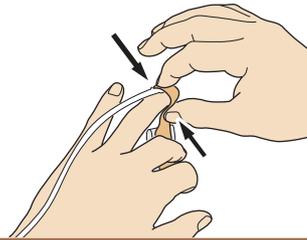

4. Remove plastic film and continue to wrap tape around finger

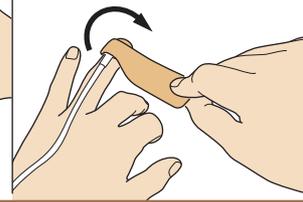


5. Adjust cable to comfortable length




Pair and Insert Chip

4


1. Hold the chip near the bedside station and listen for double beep

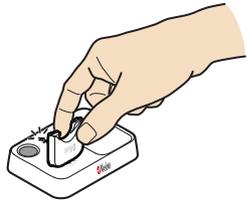


2. Insert the chip into open compartment on the sensor

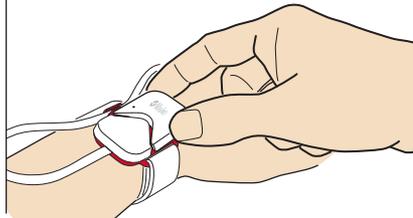

3. Press down on the chip to secure

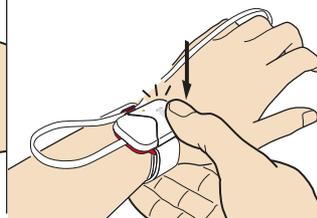

Listen for click to confirm chip placement

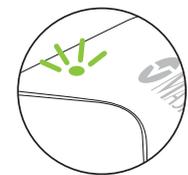

Look for light to confirm chip connection to sensor

Confirm Successful Pairing

5


1. Once the chip is inserted, the Bedside Station Bluetooth Status LED will turn off and the Status LED will turn green. The chip light indicator will turn solid blue.




Visit <http://www.mymasimo.co.uk/safetynet-alert> to see full list of supported devices, purchase additional sensors and view other resources, including videos and instructions.

Changing the Replacement Tape


1. Remove used tape from sensor

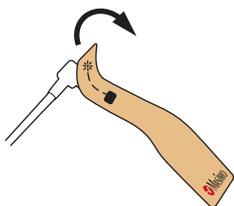

2. Attach new tape in same direction as used tape

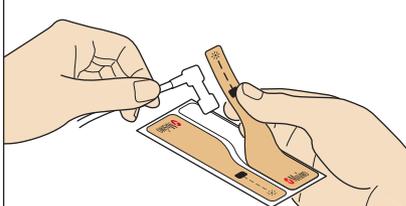

3. Attach sensor to finger

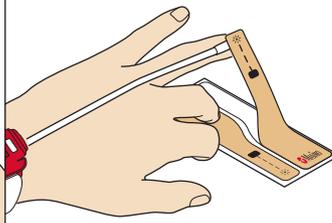

4. Peel the remaining tape off backing

